

How To Clean Your Sealed Keyboard

Your new keyboard is designed withstand repeated washings without needed to be unplugged or turning off the computer.

1. Lock your keys by pressing the left & right Control (“Ctrl”) keys together for 3 seconds, or by using the lock key on the keyboard.
2. Clean the surface of the product by wiping, brushing or spraying it using only cloths, sponges or soft bristle brushes.
3. Wipe dry with cloth or allow to air dry.
4. After cleaning, unlock keys by repeating lock process.

⚠ **DO NOT** place the USB or PS/2 connector in fluid.

RECOMMENDED CLEANING MATERIALS

- **Chlorine based (max 10%)** - Chlorox, Novalsan, ect.
- **Formaldehyde based** - Formaldehyde Solution 37%, Vinco Formaldegen, ect.
- **Glutaraldehyde based** - Aldacide 200, Lysofume, Wvacide, ect.
- **Phenol based** - Cavicide, Isopropyl Alcohol, Medicide, Dürr FD 322, FD 333 & 350, etc.
- **Oxidizing** - Hydrogen Peroxide 3% Solution, Lifeguard Series, etc.
- **Quaternary Ammonium** - Roccal-D, Bacto-Sep, Parvosol, Dürr FD 300 2%, Dürr FD 312 2%, etc.
- **Soap & Water**

⚠ **DO NOT USE**

- **Petroleum based cleaning fluids, such as Gasoline, Turpentine & oil**
- **Acetone**
- **Greater than 10% bleach solutions**
- **Abrasive cleaners**
- **Temperature or pressure sterilization**
- **Buffing pads or scrubbing pads**

These instructions & recommendations have been prepared to advise the proper methods of cleaning your premium data entry product. Man & Machine, Inc. & your local distributor cannot assume responsibility for the application of cleaners. Any questions or concerns should be addressed prior to actual cleaning by contacting Man & Machine, Inc.



The Key To Infection Control

info@mmimd.com • www.man-machine.com • Phone (301) 341-4900

